




A NEW MINDSET

*NEODENT[®] CERAMIC IMPLANT SYSTEM
WORKING PROTOCOL*

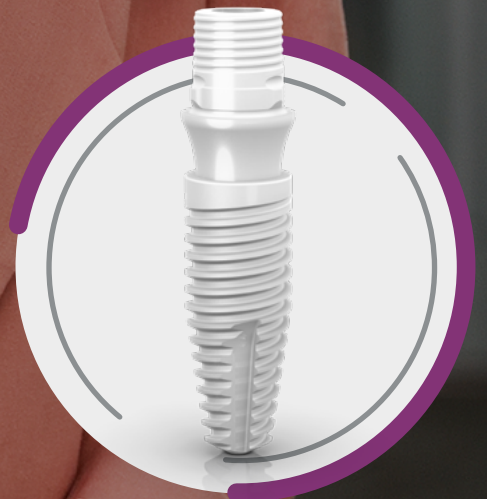

Ceramic
Implant System



ZiLock[®]
Connection

SUMMARY

| Zi Implant Ø 3.75 mm working protocol

| Zi Implant Ø 4.3 mm working protocol

| Prosthetic procedures



Ø 3.75

SURGICAL PROCEDURES



Implant lengths



- 10.0 180.002
- 11.5 180.003
- 13.0 180.004

Drills sequence for all bone type

- 1 Initial (optional) 103.170
- 2 Ø2.0 103.425
- 3 Ø3.5 103.562 short, 103.561 medium, 103.563 long
- 4 Ø3.75 103.565 short, 103.564 medium, 103.566 long

Direction Indicator



- Ø3.0/3.75
- 128.020

Drills sequence for types I, II and III bones



- 5 Countersink Ø3.75
- 103.609

Drill sequence for types I and II bones



- 6 Bone Tap Ø3.75
- 111.049

Healing phase



- Zi Cover Screw
- 117.023



- Zi Healing Abutment
- 106.233 3.75x1.5 mm
- 106.234 3.75x2.5 mm
- 106.235 4.5x1.5 mm
- 106.236 4.5x2.5 mm

Drivers and Torque Wrench



- Neo Screw Driver Manual
- 104.058 short
- 104.060 medium
- 104.059 long



- Implant Driver for Contra-Angle
- 105.002



- Implant Driver for Torque Wrench (short)
- 105.001



- Torque Wrench
- 104.050

Ø 4.3

SURGICAL PROCEDURES



Implant lengths



- 10.0 180.006
- 11.5 180.007
- 13.0 180.008

Drills sequence for all bone type

- 1 Initial (optional) 103.170
- 2 Ø2.0 103.425
- 3 Ø3.5 103.562 short, 103.561 medium, 103.563 long
- 4 Ø4.3 103.571 short, 103.570 medium, 103.572 long

Direction Indicator



- Ø3.6/4.3
- 128.022

Drills sequence for types I, II and III bones

- 5 Countersink Ø4.3 103.610

Drill sequence for types I and II bones

- 6 Bone Tap Ø4.3 111.050

Healing phase



- Zi Cover Screw 117.023



- Zi Healing Abutment
- 106.233 3.75x1.5 mm
- 106.234 3.75x2.5 mm
- 106.235 4.5x1.5 mm
- 106.236 4.5x2.5 mm

Drivers and Torque Wrench



- Neo Screw Driver Manual
- 104.058 short
- 104.060 medium
- 104.059 long



- Implant Driver for Contra-Angle 105.002



- Implant Driver for Torque Wrench (short) 105.001



- Torque Wrench 104.050

Abutment selection

Zi Base

	135.254	3.75x4x1.5 mm
	135.255	3.75x4x2.5 mm
	135.256	4.5x4x1.5 mm
	135.257	4.5x4x2.5 mm

Zi Base for C

	135.258	4.65x4.70x1.5 mm
	135.259	4.65x4.70x2.5 mm

CR Abutment

	114.854	4.0x5.0x1.5 mm
	114.855	4.0x5.0x2.5 mm
	114.856	4.5x5.0x1.5 mm
	114.857	4.5x5.0x2.5 mm

CR Abutment 17°

	114.858	4.0x5.0x1.5 mm
	114.859	4.0x5.0x2.5 mm
	114.860	4.5x5.0x1.5 mm
	114.861	4.5x5.0x2.5 mm

Impression

Implant Scanbody

	108.184
--	---------

Scanbase for C

	1.5	108.234
	2.5	108.235

Zi Implant Exact Impression Coping Open and Closed Tray

	regular	108.186
	long	108.187
		108.188
		108.189

Model production

Hybrid Repositionable Analog

	101.080
--	---------

CR Abutment Analog

	101.105	4.0x5.0 mm
	101.106	4.5x5.0 mm

Temporary

PEEK CR abutment

	114.888	4.0x5.0x1.5 mm
	114.889	4.0x5.0x2.5 mm
	114.886	4.5x5.0x1.5 mm
	114.887	4.5x5.0x2.5 mm

Impression Coping / Provisional Coping

	108.201	4.0x5.0 mm
	108.202	4.5x5.0 mm

Final coping

CARES Visual
 3Shape
 Dental Wings
 exocad Exocad

Burn-out Coping

	118.343	3.75x4 mm
	118.325	4.5x4 mm

CR Abutment Burn Out coping

	118.367	4.0x5.0 mm
	118.368	4.5x5.0 mm

Drivers

Neo Screw Driver Manual

	104.058	short
	104.060	medium
	104.059	long

Neo Screwdriver torque wrench

	105.133	short
	105.132	medium
	105.134	long

© 2022 – JIGC Indústria e Comércio de Materiais Dentários S.A. All rights reserved. Neodent®, Zi™ and ZiLock® are trademarks or registred trademarks of JIGC Indústria e Comércio de Materiais Dentários S.A. CARES is a trademark or registered trademark of Straumann Holding AG. 3Shape is a trademark or registered trademark of 3Shape A/S. exocad is a trademark or registered trademark of exocad GmbH. Dental Wings is a trademark or registered trademark of Dental Wings Inc. The reproduction of this material without prior authorization is prohibited. Some products may not be available for purchase yet. For more informations, please contact your local distributor.

10475_neodent_zi_workingprotocol_en_xx_E00_of_180723

毎日、とびっきりの笑顔を

NUOVI SORRISI OGNI GIORNO

CHAQUE JOUR DE NOUVEAUX SOURIRES

новые улыбки каждый день

JEDEN TAG EIN NEUES LÄCHELN

NUEVAS SONRISAS TODOS LOS DÍAS

NYA LEENDEN VARJE DAG

NEW SMILES EVERY DAY

NOVOS SORRISOS TODOS OS DIAS

CHAQUE JOUR DE NOUVEAUX SOURIRES

NOVOS SORRISOS TODOS OS DIAS



www.neodent.com